

MASK

Model No.:175*95

Standard :EN 149:2001+A1:2009

Type I



**Ganzhou Zhongke TUOYOU DA Intelligent
Equipment Technology Co.,Ltd**

Room A01,10th Floor,JinZhou

International business center,No.899,Jiefang North
Road,Yuexiu District,Guangzhou City

[Specification type] 17.5*9.5cm,3 layers mask, elastic
rope for ear;metal strips for nose wire

[Product protection level] II

[Instruction for using]

- 1 .Nose clip up, hold one ear curtain with both hands;
- 2.Put on the mask, put the chin in the mask, buckle the
ear bag behind the ear with both hands, and make the
mask stick to the face;
- 3.Use the forefinger and middle finger of both hands to
press and adjust the nasal clamp to stick to the bridge
of the nose.

[Be attention] If dizziness,nausea and dyspnea occur
during use, please replace the product.

This product is disposable and will be destroyed after
use.

[Notice] The enterprise can resume production
of the daily protective mask, and can not be used in
any medical institutions and industries with special
requirements for protection.

[Storage]

In a dry,ventilated, non hazardous environment.

Lot:2020 Mar 1st

Expired date:2025 Mar 1st

Package:10pcs

Distributor:OU DA INTERNATIONAL LIMITED